M&bility 4Ki&s

Welcome to Mobility 4 Kids—an
Innovative Products Inc. manufacturing
company that specializes in custom
built mobility devices for children with
disabilities. We at Mobility 4 Kids pride
ourselves on offering our customers
superior custom built quality products
and service. If there is a specific mobility
device that you need, we welcome
questions at anytime.

M&bility 4 Kids

830 S. 48th St. Grand Forks, ND, 58201

Toll Free: 800.950.5185 Office Phone: 701.772.5185 Cell Phone: 218.779.0158 Fax: 701.772.5284

For demonstration videos or Powerpoint presentations, please visit:

www.mobility4kids.com
www.bowlingramps.com
www.lPhope.com






Mcbility 4 Kids



LARGE BOAR

Allows anyone to stand who can bear weight

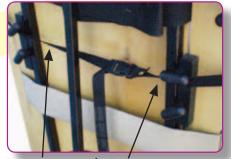



On/Off Switch for Electronic Controls



Manual Switch to Move Overboard (Up/Down)



P&G Omni Interface for Switches

Large Power Board Specifications:

- Made of ³/₄" thick plywood x 28" wide x 6 foot tall
- Continuous slots running up and down the board for easy adjustment
- Lift up and swing away arm/hip supports with 2" foam hip supports
- Chest, Hip, Knee straps
- Quick removable tray (3/8"x 28"x 24")
- Anti tip support post
- 24V Power DC actuator to go from laying to full standing (switch operated)
- Lifting capacity of 450lbs
- 24 V P&G electronics
- Center pivot drive wheels (turns 360°)

How the Power Board Works:

- Use a transfer board to go from the wheelchair to the power board
- · Lay adult or child on their back temporarily (be sure to lay a small pillow under their head)
- Tighten the chest, hip and knee straps
- Start raising the power board either with the optional switch provided on the back of the board or by the joystick
- Stand the power board in a vertical position, adjust the arms, install the tray, and now drive the board around either with a joystick or single switches



Optional Bars to KneeBlocks for Small Children



Benefits:

- Allows standing
- Easy to position a person from horizontal laying position to full standing
- Power base can be operated from several options (joystick, switches, sip & puff switch, and many others)

Optional Equipment Available:

- P&G R-Net or Omni Switch Interface
- Able Net CA-5 Switch Interface
- Single Switches
- Head arrays

Schools can buy



From Manufacturer. Funding programs available for schools.

