



Welcome to Mobility 4 Kids—an Innovative Products Inc. manufacturing company that specializes in custom built mobility devices for children with disabilities. We at Mobility 4 Kids pride ourselves on offering our customers superior custom built quality products and service. If there is a specific mobility device that you need, we welcome questions at anytime.

830 S. 48th St.  
Grand Forks, ND, 58201

Toll Free: 800.950.5185  
Office Phone: 701.772.5185  
Cell Phone: 218.779.0158  
Fax: 701.772.5284

For demonstration videos  
or Powerpoint presentations,  
please visit:  
[www.mobility4kids.com](http://www.mobility4kids.com)  
[www.bowlingramps.com](http://www.bowlingramps.com)  
[www.IPhope.com](http://www.IPhope.com)



All products manufactured  
by Innovative Products, Inc.



# LARGE POWER BOARD

Allows anyone to stand who can bear weight

## Large Power Board Specifications:


- Made of ¾" thick plywood x 28" wide x 6 foot tall
- Continuous slots running up and down the board for easy adjustment
- Lift up and swing away arm/hip supports with 2" foam hip supports
- Chest, Hip, Knee straps
- Quick removable tray (3/8"x 28"x 24")
- Anti tip support post
- 24V Power DC actuator to go from laying to full standing (switch operated)
- Lifting capacity of 450lbs
- 24 V P&G electronics
- Center pivot drive wheels (turns 360°)

## How the Power Board Works:

- Use a transfer board to go from the wheelchair to the power board
- Lay adult or child on their back temporarily (be sure to lay a small pillow under their head)
- Tighten the chest, hip and knee straps
- Start raising the power board either with the optional switch provided on the back of the board or by the joystick
- Stand the power board in a vertical position, adjust the arms, install the tray, and now drive the board around either with a joystick or single switches

## Benefits:

- Allows standing
- Easy to position a person from horizontal laying position to full standing
- Power base can be operated from several options (joystick, switches, sip & puff switch, and many others)

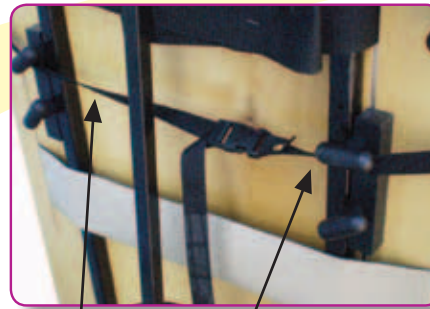

Tie Down Location for Tray



On/Off Switch for Electronic Controls



Optional Bars to KneeBlocks for Small Children



Manual Switch to Move Overboard (Up/Down)



Switch Interface



P&G Omni Interface for Switches

## Optional Equipment Available:

- P&G R-Net or Omni Switch Interface
- Able Net CA-5 Switch Interface
- Single Switches
- Head arrays

**Schools  
can buy  
direct!**

**SCHOOLS PURCHASE DIRECT!**

From Manufacturer. Funding programs available for schools.

1.800.950.5185

**Mobility  
4 Kids**